

Full-Field Pupillometry

ColorDome Upgrade Module


The Full-field Pupillometry module works with the ColorDome™ camera to monitor pupil size of both eyes and display video of the patient's eyes in the Espion™ software during and after a flash.

Features

- Uses the original ColorDome camera
- Uses infrared LED illumination
- Simple set-up: no need for a headband
- Pupil response synchronized with trigger to the ColorDome flash
- Records from any ColorDome color flash stimulus
- Records area or diameter of pupils
- Both eyes stimulated and recorded simultaneously
- Image displayed in Espion software



ColorDome shown mounted on table stand

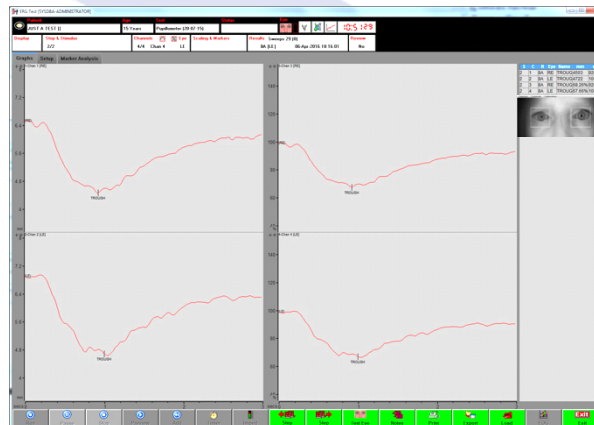

Software showing the pupil response to a flash in absolute diameter and relative to a baseline

Full-field Pupillometry module includes:

-
- LiveTrack AP
 - BNC connector to phono connector
 - Trigger cable (to connect to trigger out on Espion)
 - Calibration card
 - USB cable
 - Espion Pupillometer protocols
-




Application examples:

The pupillary light reflex (PLR) has been shown to be a valuable clinical test in assessing the health of the rod, cone and melanopsin pathways^{1,2}. One of the standard protocols offered with the Diagnosys pupillometer measures the PLR for each of these pathways.

Summary:

1. Rod pathway: 0.001 cd/m² blue on 0 cd/m² background, for 1 second (after 10 min dark adapt)
2. Red cone pathway: 20 cd/m² red on 0.1 cd/m² blue background, for 1 second
3. All cone pathways: 3 cds/m² white 4 ms flash on 1.0 cd/m² white background
4. Melanopsin pathway: 150 cd/m² blue on 0 cd/m² background, for 1 second

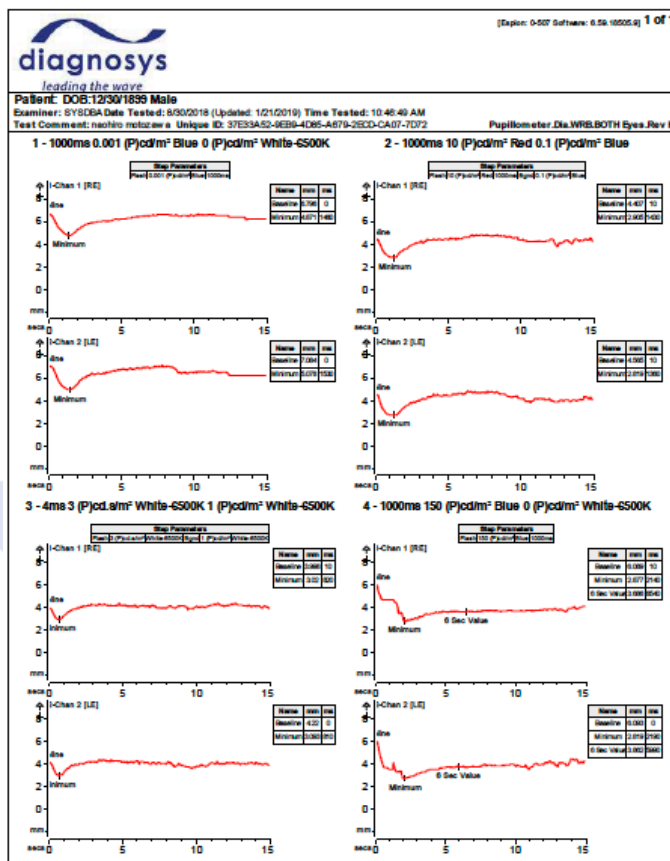

Pupillometry module ordering information

Model # D355

Available as an upgrade option to any ColorDome, on the following systems:

- Diagnosys *E*³ desktop or *Profile* cart systems (with Espion software Ver 6.61+)

1. Hood, DC, et al; "Toward a Clinical Protocol for Assessing Rod, Cone, and Melanopsin Contributions to the Human Pupil Response;" Investigative Ophthalmology & Visual Science, August 2011, Vol. 52, No. 9
2. Stone, EM, et al; "Full-Field Pupillary Light Responses, Luminance Thresholds, and Light Discomfort Thresholds in CEP290 Leber Congenital Amaurosis Patients;" Invest Ophthalmol Vis Sci. 2015;56:7130-7136. DOI:10.1167/iops.15-17467
3. Collison, FT, et al; "Two-color pupillometry in KCNV2 retinopathy;" Doc Ophthalmol (2019) 139:11-20

www.diagnosysllc.com

US: Diagnosys LLC; 55 Technology Drive, Suite 100, Lowell, MA 01851; 978-458-1600; sales@diagnosysllc.com

EU: Diagnosys Vision Ltd; Office 117, DOC Building, Balheary Road, Swords, Dublin, K67 E5A0, Ireland; +44 (0) 1223 520699; mail@diagnosysvision.com

UK: Diagnosys UK Ltd; 5 Trust Court, Chivers Way, Vision Park Histon, Cambridge, CB24 9PW, UK; +44 (0) 1223 520699; mail@diagnosysuk.co.uk

The information listed herein is accurate as of the date of printing, however may change at any time without notice. The contents may differ from the current status of approval of the product in your country. Please contact your local Diagnosys representative for more information. © 2021 by Diagnosys, LLC. All rights reserved.

Doc: 14679 Rev: F ECN 1605 Date: 15 Feb 2021