



The DiagnosisFST®, Full-field Stimulus Threshold, module uses the ColorDome™ or ColorFlash™ stimulator. It measures the sensitivity of the visual field by testing for the lowest luminance flash which elicits a visual sensation perceived by the subject. The test is run on either dark- or light-adapted patients for one or both eyes in an automated routine to measure a reliable threshold.

Applications

- Gene therapy diagnostics
- Safety or primary endpoint in clinical trials
- Clinical low vision patient objective measure of vision
- RP, LCA, Stargardt disease, CSNB, Achromatopsia, Choroideremia, retinal prosthesis, others
- 15 years of clinical trial success

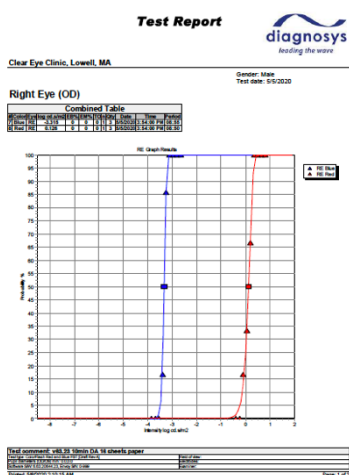

Dark adapted Red & Blue: Low vision patient

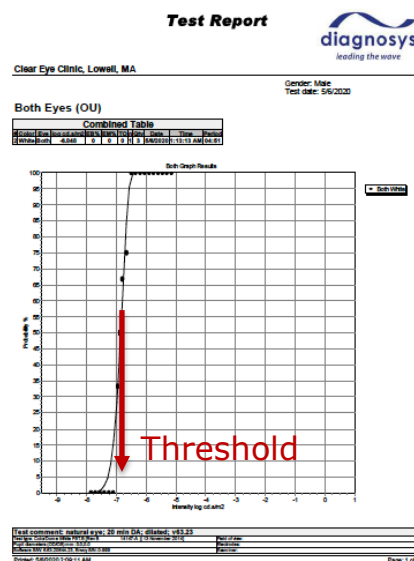

Dark adapted White: normal patient threshold

ColorDome and ColorFlash stimulator options:



ColorDome



ColorDome with iMask




ColorFlash¹

1. ColorFlash stimulus range is appropriate for FST tests on low vision patients only.

Why use DiagnosysFST®:

Roman, et al in 2007:

1. Test ultra-low vision patients with a quantified, repeatable measure of vision
2. Up through at least 'barely light perception'
3. Patients with vision up to 60 dB (6 log units in $\text{cd}\cdot\text{s}/\text{m}^2$) worse vision than Normal subjects were repeatably measured

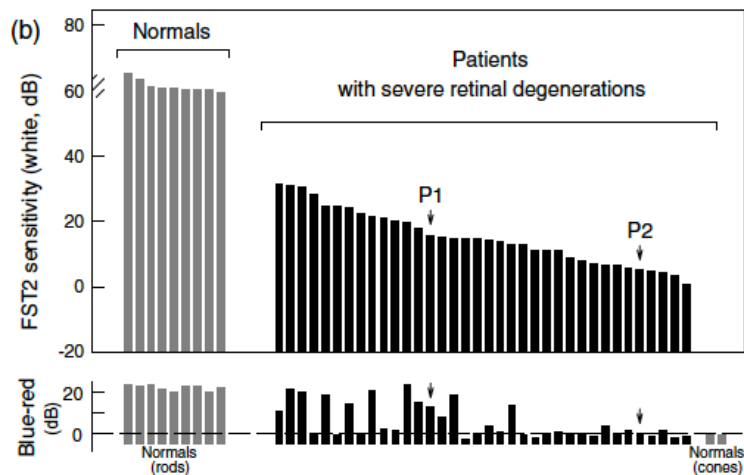

DiagnosysFST® proven test-retest repeatability:

Klein and Birch in 2009:

1. 42 low vision subjects (no ERG response; unable to perform static perimetry)
2. ± 2 standard deviations (95% CI) was found to be ± 3 dB (± 0.3 Log $\text{cd}\cdot\text{s}/\text{m}^2$)


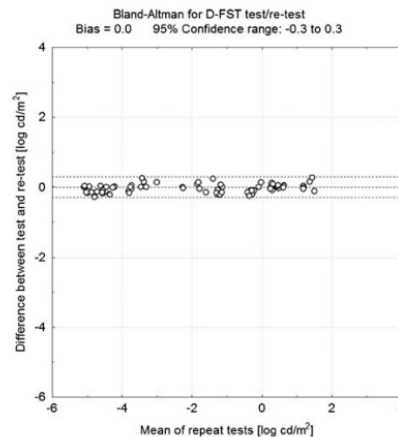


Fig. 4. Bland-Altman analysis for the test/retest variability of the D-FST. The difference in threshold between test and retest within a session (y-axis) is plotted against the mean threshold for all sessions (x-axis). The 95% confidence range (dashed lines) is ± 0.3 log cd/m^2


Relied on for over a decade of Gene Therapy clinical trials:

Albert M. Maguire, et al; 'Efficacy, Safety, and Durability of Voretigene Neparvovec-rzyl in RPE65 Mutation - Associated Inherited Retinal Dystrophy'; 2019


Mean **Multi-Luminance Mobility Test (MLMT)** lux scores from administration baseline in evaluable phase 1 follow-on subjects (n = 8) and phase 3 subjects.

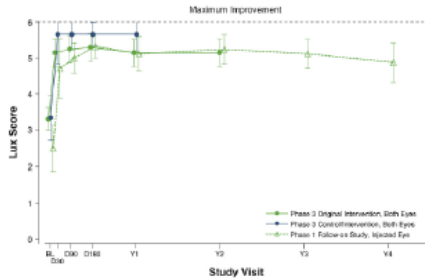

Mean **full-field light sensitivity threshold (FST)**, white light, averaged over both eyes, from administration baseline in evaluable phase 1 follow-on subjects (n = 8) and phase 3 subjects.

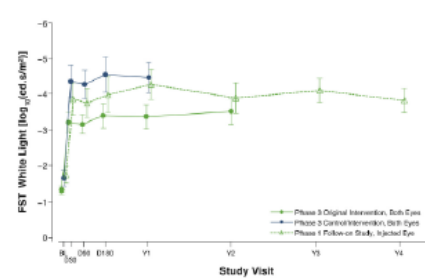

Samuel G. Jacobson et al; 'Defining Outcomes for Clinical Trials of Leber Congenital Amaurosis Caused by GUCY2D Mutations'; 2017

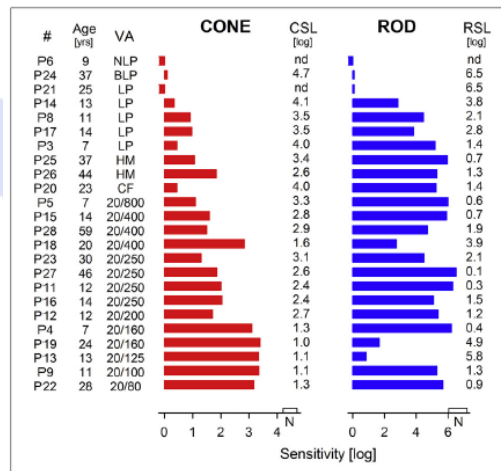

Benchmarked against all other diagnostic tests

Multi-Luminance Mobility Test (MLMT), other courses	Maguire 2019; Jacobson 2017; Russell 2017; Bennett 2016; Jacobson 2012
Visual fields, perimetry	Stingl 2019; Aleman 2018; Utz 2018; Stunkel 2018; Dimopoulos 2017; Ghazi 2016; Bennett 2016; Collison 2014; Bittner 2014; Messias 2013; Jacobson 2012; Lorenz 2012; Jacobson 2011; Cideciyan 2011
OCT	Aleman 2018; Utz 2018; Stunkel 2018; Jacobson 2017; Ghazi 2016; Collison 2014; Ahuja 2013; Jacobson 2012; Jacobson 2009; Hauswirth 2008
ERG	Stingl 2019; Utz 2018; Stunkel 2018; Collison 2014; Messias 2013; Cideciyan 2011; Klein 2009; Jacobson 2009
Pupillometry	Stingl 2019; Jacobson 2017; Collison 2015; Collison 2014; Jacobson 2012; Lorenz 2012; Jacobson 2011; Maguire 2009
Fundus photography, autofluorescence	Dimopoulos 2017; Jacobson 2017; Ahuja 2013
Contrast sensitivity	Bittner 2014

Thoroughly studied in peer-reviewed papers (select list):

1. Alejandro J Roman, et al; 'Full-field stimulus testing (FST) to quantify visual perception in severely blind candidates for treatment trials'; *Physiol. Meas.* 28 (2007) N51–N56. doi:10.1088/0967-3334/28/8/N02
2. William W. Hauswirth, et al; 'Treatment of Leber Congenital Amaurosis Due to RPE65 Mutations by Ocular Subretinal Injection of Adeno-Associated Virus Gene Vector: Short-Term Results of a Phase I Trial'; *HUMAN GENE THERAPY* 19:979–990 (October 2008). doi: 10.1089/hum.2008.107
3. Albert M Maguire, et al; 'Age-dependent effects of RPE65 gene therapy for Leber's congenital amaurosis: a phase 1 dose-escalation trial'; *Lancet.* 2009 November 7; 374(9701): 1597–1605. doi:10.1016/S0140-6736(09)61836-5.
4. Samuel G. Jacobson, et al; 'Defining the Residual Vision in Leber Congenital Amaurosis Caused by RPE65 Mutations'; *Invest Ophthalmol Vis Sci.* 2009 May; 50(5): 2368–2375. doi:10.1167/iovs.08-2696
5. M Klein and DG Birch; 'Psychophysical assessment of low visual function in patients with retinal degenerative diseases (RDDs) with the Diagnosys full-field stimulus threshold (D-FST)'; *Doc Ophthalmol.* 2009 December ; 119(3): 217–224. doi:10.1007/s10633-009-9204-7
6. Artur V. Cideciyan, et al; 'Cone photoreceptors are the main targets for gene therapy of NPHP5 (IQCB1) or NPHP6 (CEP290) blindness: generation of an all-cone Nphp6 hypomorph mouse that mimics the human retinal ciliopathy'; *Human Molecular Genetics*, 2011, Vol. 20, No. 7. doi:10.1093/hmg/ddr022
7. Samuel G. Jacobson et al; 'Human Retinal Disease from AIPL1 Gene Mutations: Foveal Cone Loss with Minimal Macular Photoreceptors and Rod Function Remaining'; *Investigative Ophthalmology & Visual Science*, January 2011, Vol. 52, No. 1. doi:10.1167/iovs.10-6127
8. Samuel G. Jacobson et al; 'Gene Therapy for Leber Congenital Amaurosis caused by RPE65 mutations: Safety and Efficacy in Fifteen Children and Adults Followed up to Three Years'; *Arch Ophthalmol.* 2012 January ; 130(1): 9–24. doi:10.1001/archophthalmol.2011.298
9. Birgit Lorenz, et al; 'Chromatic Pupillometry Dissects Function of the Three Different Light-Sensitive Retinal Cell Populations in RPE65 Deficiency'; *Invest Ophthalmol Vis Sci.* 2012;53:5641–5652 DOI:10.1167/ iovs.12-9974
10. AK Ahuja, et al; 'Factors Affecting Perceptual Threshold in Argus II Retinal Prosthesis Subjects'; *Trans Vis Sci Tech.* 2013;2(4):1. doi/full/10.1167/tvst.2.4.1
11. Katharina Messias, et al; 'Psychophysically determined full-field stimulus thresholds (FST) in retinitis pigmentosa: relationships with electroretinography and visual field outcomes; *Doc Ophthalmol* (2013) 127:123–129. doi:10.1007/s10633-013-9393-y
12. Ava K Bittner, et al; 'A pilot study of an acupuncture protocol to improve visual function in retinitis pigmentosa patients'; *Clin Exp Optim.* 2014 May; 97(3): 240–247. doi:10.1111/cxo.12117
13. Frederick T. Collison, et al; 'Psychophysical Measurement of Rod and Cone Thresholds in Stargardt Disease with Full-Field Stimuli'; *Retina.* 2014 September ; 34(9): 1888–1895. doi:10.1097/IAE.000000000000144
14. Frederick T. Collison, et al; 'Full-Field Pupillary Light Responses, Luminance Thresholds, and Light Discomfort Thresholds in CEP290 Leber Congenital Amaurosis Patients'; *Invest Ophthalmol Vis Sci.* 2015;56:7130–7136. doi:10.1167/iovs.15-17467
15. Nicola G. Ghazi, et al; 'Treatment of retinitis pigmentosa due to MERTK mutations by ocular subretinal injection of adeno-associated virus gene vector: results of a phase I trial'; *Hum Genet*, doi:10.1007/s00439-016-1637-y
16. Jean Bennet, et al; 'Safety and durability of effect of contralateral-eye administration of AAV2 gene therapy in patients with childhood-onset blindness caused by RPE65 mutators: a follow-on phase 1 trial'; *Lancet.* 2016 August 13; 388(10045): 661–672. doi:10.1016/S0140-6736(16)30371-3
17. Samuel G. Jacobson et al; 'Defining Outcomes for Clinical Trials of Leber Congenital Amaurosis Caused by GUCY2D Mutations'; *AmJOPhthalmol* 2017;177:44–57. <http://dx.doi.org/10.1016/j.ajo.2017.02.003>
18. Stephen Russell, et al; 'Efficacy and safety of voretigene neparvovec (AAV2-hRPE65v2) in patients with RPE65-mediated inherited retinal dystrophy: a randomised, controlled, open-label, phase 3 trial'; *Lancet* 2017; 390: 849–60. doi: [http://dx.doi.org/10.1016/S0140-6736\(17\)31868-8](http://dx.doi.org/10.1016/S0140-6736(17)31868-8)
19. Samuel G. Jacobson, et al; 'Outcome Measures for Clinical Trials of Leber Congenital Amaurosis Caused by the Intronic Mutation in the CEP290 Gene'; *Invest Ophthalmol Vis Sci.* 2017;58:2609–2622. DOI:10.1167/iovs.17-21560
20. Ioannis S Dimopoulos, et al; 'The Natural History of Full-Field Stimulus Threshold Decline in Choroideremia'; *RETINA* 0:1–12, 2017.
21. Maria L. Stunkel, et al; 'Expanded retinal disease spectrum associated with autosomal recessive mutations in GUCY2D'; *AJOPHT* 10455. doi:10.1016/j.ajo.2018.03.021
22. Virginia Miraldi Utz, et al; 'Presentation of TRPM1-Associated Congenital Stationary Night Blindness in Children'; *JAMA Ophthalmol.* doi:10.1001/jamaophthalmol.2018.0185
23. Tomas S Aleman, et al; 'RDH12 Mutations Cause a Severe Retinal Degeneration With Relatively Spared Rod Function'; *Invest Ophthalmol Vis Sci.* 2018;59:5225–5236. <https://doi.org/10.1167/iovs.18-24708>
24. Krunoslav T. Stingl, et al; 'Chromatic Full-Field Stimulus Threshold and Pupillography as Functional Markers for Late-Stage, Early-Onset Retinitis Pigmentosa Caused by CRB1 Mutations'; *Trans Vis Sci Tech.* 2019;8(6):45, <https://doi.org/10.1167/tvst.8.6.45>
25. Artur V. Cideciyan, et al; 'Effect of an intravitreal antisense oligonucleotide on vision in Leber congenital amaurosis due to a photoreceptor cilium defect'; *Nature Medicine*, VOL 25, FEBRUARY 2019, 225–228. doi:10.1038/s41591-018-0295-0
26. Albert M. Maguire, et al; 'Efficacy, Safety, and Durability of Voretigene Neparvovec-rzyl in RPE65 Mutation - Associated Inherited Retinal Dystrophy'; *Ophthalmology* 2019;126:1273-1285. doi:10.1016/j.ophtha.2019.06.017

www.diagnosysllc.com

US: Diagnosys LLC; 55 Technology Drive, Suite 100, Lowell, MA 01851; 978-458-1600; sales@diagnosysllc.com

EU: Diagnosys Vision Ltd; Office 117, DOC Building, Balheary Road, Swords, Dublin, K67 E5A0, Ireland; +44 (0) 1223 520699; mail@diagnosysvision.com

UK: Diagnosys UK Ltd; 5 Trust Court, Chivers Way, Vision Park Histon, Cambridge, CB24 9PW, UK; +44 (0) 1223 520699; mail@diagnosysuk.co.uk

The information listed herein is accurate as of the date of printing, however may change at any time without notice. The contents may differ from the current status of approval of the product in your country. Please contact your local Diagnosys representative for more information. © 2021 by Diagnosys, LLC. All rights reserved.

Doc: 15925 **Rev:** B ECN 1605 **Date:** 15 Feb 2021